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PREPARATION OF NEBULISER (FIG. 1)
- Unscrew the nebuliser.
- Fill in drug(s).
- Screw back again the nebuliser.

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OXYGEN SUPPLY (FIG. 2)
- Connect the nebuliser extension tube (translucent) on oxygen bottle
- Connect the O₂ flow meter 0-30L/min.
- Position it vertically.
- Connect the CPAP green extension tube to the flow meter.

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ONLINE OF THE MANOMETER (FIG. 3)
- Pass the manometer attachment cord around patient's neck.
- Connect the manometer attachment cord to one of the PEEP pressure gauge wings.
- Insert the manometer tubing firmly into the PEEP pressure gauge.
- Attach the other side of the manometer tubing to the clear translucent port of the CPAP (fig. 4).

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CHOICE OF THE MASK (FIG. 5)
- Position patient with mouth half open.
- Define the distance between the nose root and the nasal-chin furrow.
- Choose the best mask according to this distance.
- Inflate the cuff.
The mask's width must cover the mouth.

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SETTING UP THE CPAP/NEBULISER ON MASK (FIG. 6)
- Connect nebuliser extension under nebuliser (6.1)
- Connect "T" piece over nebuliser (6.2)
- Connect "T" piece & nebuliser to the face mask (6.3)
- Connect the other side of "T" piece to the CPAP (6.4).

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SETTING UP THE CPAP/NEBULISER ON A MONITORED PATIENT
- Nebuliser: open the flow meter at 6L/min. (fig. 7).
- CPAP: open the flow meter at 10L/min. (fig. 8).
- Explain the procedure to the patient.
- Apply the CPAP/nebuliser to patient's face with one hand and put the other hand behind the patient's head. (fig. 9).
- When the patient is tolerating the mask, position the harness (with the help of a second person if necessary). (fig. 10).
Check there is no leakage.
- Modulate the flow rate to obtain pressure level 2 to 4 cm H₂O. (fig. 11).

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ADJUSTMENT AND MONITORING
- Check there is no:
• Leakage (ask patient/hand around mask)
• Areas where the mask is pressed too tightly on the patient's face.
- You could modify mask position:
• Inflate or deflate the cuff.
• Change the harness traction
- Maintain clinical monitoring of the patient.
If there is a pressure decrease: check for: leakage (mask/extension tubing) whether the oxygen bottle is empty.
Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

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