




Boussignac CPAP System with Nebuliser

Vygon's Effective Tilttable Drug Delivery System

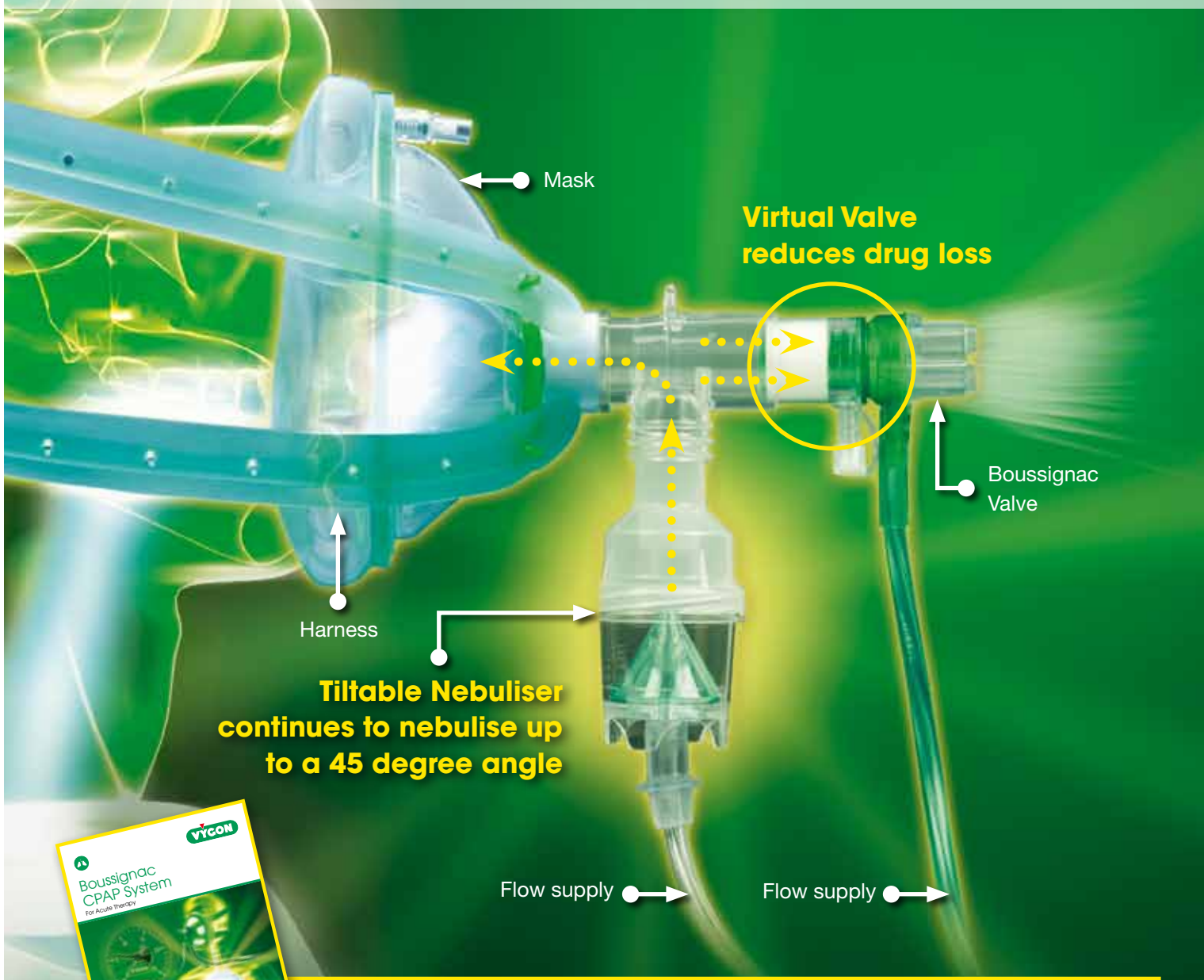

Mask

**Virtual Valve
reduces drug loss**

Boussignac
Valve

Harness

**Tilttable Nebuliser
continues to nebulise up
to a 45 degree angle**

Flow supply

Flow supply



For further information on Boussignac, please refer to the 'Boussignac CPAP System' literature, available on request from your local Vygon Sales Executive


vygon@vygon.co.uk

www.vygon.co.uk

Nebuliser Cross Section

Easy assembly/disassembly with screw-on cap provides a leak-proof seal

Greater surface area creates maximum capillary action. Allows operation at any angle

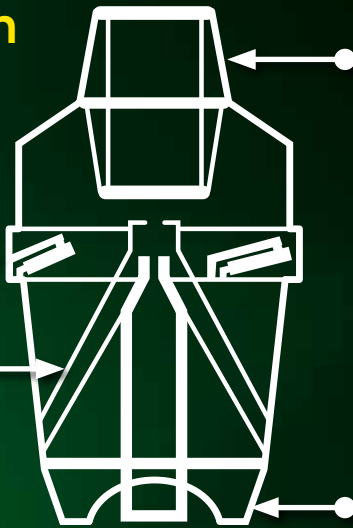

Top fits either a tee or an aerosol mask

Scalloped cuff provides stability and simplifies tube attachment

Features and Benefits

Combined Therapy

- No loss of PEEP
- Decrease work of breathing
- Increase bronchodilatation
- Decrease airway resistance
- Re-expand atelectasis
- Offset intrinsic PEEP

Boussignac with Humidification

- Comfortable for patient
- Tolerated longer

Loss of Drug Reduced

- More absorbed by patient
- Quicker recovery (Higher PEF)

Tiltable Nebuliser

- Used directly on face mask
- Continues to nebulise up to a 45 degree angle

Directions for Use

1. Select the appropriate size mask kit
2. Fill nebuliser with drug
3. Connect nebuliser to flow supply
4. Attach 30L/min flow meter and attach Boussignac Valve
5. Attach nebuliser to Boussignac Valve
6. Attach PEEP tubing to Boussignac Valve and PEEP Gauge
7. Attach assembly to face mask
8. Turn nebuliser on to 6L/min first, then turn on flow to Boussignac Valve low at first 10-12L/min
9. Gradually place over patient until tolerable
10. Attach harness
11. Turn up flow on Boussignac Valve until desired PEEP is achieved
12. Remove PEEP tubing and cap monitoring port

Ordering Information

Product Code	Description and Contents	Box Quantity
5572.403	Boussignac CPAP Small Adult Set + Nebuliser 1 x Valve, 1 x Mask size 4, 1 x Harness & Nebuliser	5
5572.503	Boussignac CPAP Medium Adult Set + Nebuliser 1 x Valve, 1 x Mask Size 5, 1 x Harness & Nebuliser	5
5572.603	Boussignac CPAP Large Adult Set + Nebuliser 1 x Valve, 1 x Mask Size 6, 1 x Harness & Nebuliser	5
5569.01	Nebuliser	10

