LEADER-CATH LEADER-FLEX

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Instructions for use

PRODUCT DESCRIPTION

Leader-Cath and Leader-Flex catheters, described below, are recommended for :

- arterial catheterisation in adults (see Part I)

- venous catheterisation (jugular, subclavian) in children (see Part II)

They are supplied in a rigid, transparent tubular pack (except REF 1212) containing :

- 1 catheter with winged hub for skin fixation,

- 1 introducer needle,

- 1 straight guidewire.

REF	Transparent PE catheter with XRO lines				Straight guidewire		Needle		
	Int. Ø - Ext. Ø mm	G	Length cm	Flow rate ml/min	Int. Ø mm	Length cm	Int. Ø - Ext. Ø mm	G	Length mm
115.09	0,6-0,9	20	8	24	0,5	20	0,6-0,9	20	32
115.11	0,8-1,2	18	10	47	0,7	30	0,8-1,1	19	54

REF	XRO catheter (PUR)				Straight guidewire		Needle		
	Int. Ø - Ext. Ø mm	G	Length cm	Flow rate ml/min	Int. Ø mm	Length cm	Int. Ø - Ext. Ø mm	G	Length mm
1211.04	0,6-0,9	20	4	30	0,5	20	0,6-0,9	20	32
1211.06	0,6-0,9	20	6	27	0,5	20	0,6-0,9	20	32
1211.08	0,6-0,9	20	8	24	0,5	20	0,6-0,9	20	32
1211.12	0,6-0,9	20	12	20	0,5	30	0,6-0,9	20	32
1212.04	0,5-0,7	22	4	17	0,5	20	0,5-0,8	21	42
1212.06	0,5-0,7	22	6	14	0,5	20	0,5-0,8	21	42
1212.08	0,5-0,7	22	8	12	0,5	20	0,5-0,8	21	42
1213.06	0,8-1,2	18	6	55	0,7	30	0,8-1,1	19	54
1213.10	0,8-1,2	18	10	44	0,7	30	0,8-1,1	19	54

I - ARTERIAL CATHETERISATION

Note : The products more specifically indicated for arterial catheterisation are: Leader-Cath REF 115.09 and 115.11 and Leader-Flex REF 1211.04, 1211.06, 1212.06 and 1213.06.

Other, longer models of the range can also be used, e.g. REF 115.12 for a femoral artery. These catheters are described in detail in the Vygon catalogue. However, their use is identical to that described below.

The approaches generally used for invasive blood pressure monitoring are : preferably the radial artery, but also the dorsalis pedis, femoral, brachial and axillary arteries.

IMPORTANT CAUTIONS

1) *Before inserting a catheter into the radial artery, perform Allen's test to confirm the presence of a good blood supply to the patient's hand if the radial artery were to become thrombosed.*

2) *The arterial puncture site must be maintained in such a way that it can be frequently and easily examined.*

3) *To avoid confusion between venous and arterial lines it is recommended that arterial lines should be colour coded and/or labelled appropriately.*

4) *Regularly check all connections of the system.*

5) *The entire system (dome of the pressure transducer, extension tube, etc .) must be permanently filled with heparinised isotonic saline solution and must not contain any air bubbles or blood.*

6) *Check the patency of the catheter at least every three hours by delivering a bolus of heparinised isotonic saline via the flow regulator.*

7) *Check correct blood reflux at least twice every shift.*

8) *Never inject drugs via an arterial catheter.*

9) *All staff should adopt "universal precautions" against needle stick injury and contact with blood and associated risks from HIV (Human Immunodeficiency Virus), Hepatitis infections, etc.*

10) *Do not expose these devices to contact with alcohol, acetone or solvent based disinfectants or dressing sprays. Particular care should be taken to avoid such contact with the hub of the Seldinger needle and with all junctions or connection points e.g. catheter hub.*

A SUGGESTED INSERTION PROCEDURE (radial artery)

1) *Check pack for contents and packaging for integrity.*

2) *Use aseptic techniques during the introduction and use of this product*

3) *Before inserting a catheter into the radial artery, perform Allen's test to confirm the presence of a good blood supply to the patient's hand if the radial artery were to become thrombosed.*

4) *Install the patient's arm on an armboard at body height with the wrist in slight extension.*

5) *Shave and disinfect the skin of the wrist. Palpate the artery at two points to determine its position and direction.*

6) *Inject local anaesthetic on either side of the artery.*

Caution

a) *The injection of local anaesthetic may make it more difficult to detect the radial pulse. Gentle massage at the infiltration site helps to restore a palpable pulse.*

b) *It may sometimes be necessary to make a 1 mm skin incision to facilitate insertion of the catheter. However, great care must be taken to avoid damaging the artery.*

7) *Maintain the artery between two fingers and puncture it with the needle.*

8) *When blood reflux is observed, immobilise the needle and introduce the flexible tip of the guidewire through the needle into the artery, advancing it about 10 centimetres.*

Caution

The guidewire should advance without resistance. If resistance is felt, do not persist, stop and withdraw both the needle and guidewire simultaneously. Never pull back or withdraw the guidewire through the needle as this could damage the guidewire on the needle bevel.

9) *Apply pressure to the artery with a finger and carefully remove the needle, leaving the guidewire in place.*

Caution

Before the catheter is advanced through the skin, check that there is sufficient length of guidewire exposed to ensure that the guidewire will exit the catheter hub.

10) *Hold the catheter close to the skin and advance it over the guidewire into the artery by applying a light twisting motion.*

11) *When the catheter is in place, carefully remove the guidewire.*

12) *Connect a 10 to 30 cm extension tube (e.g. Lectro-Cath) and a 3-way stopcock filled with isotonic saline.*

13) *Connect a syringe, half-filled with isotonic saline, to the stopcock and check patency of the line by aspirating blood into the syringe, then flush the catheter and the extension tube.*

14) *After closing the stopcock and removing the syringe, the catheter hub is sutured to the skin.*

15) *Connect the flow regulator to the 3-way stopcock, open the stopcock and flush the catheter by activating the flow regulator valve.*

16) *Apply the dressing. Anchor the flow regulator and extension tube to the skin.*

REMOVAL PROCEDURE

The removal of an arterial catheter requires special attention to the exit site. Following removal direct firm pressure is needed for at least 10 to 15 minutes over the vessel, to prevent the formation of a haematoma. The site should then be observed closely and further pressure applied if needed.

11- JUGULAR VEIN OR SUBCLAVIAN VEIN CATHETERISATION IN CHILDREN Important cautions

The using physician should be familiar with and undertake all standard precautions to avoid problems or complications associated with the introduction of venous catheters using the Seldinger technique.

Complications associated with percutaneous catheter placement include vessel wall perforation, cardiac tamponade, haemorrhage, pleural and mediastinal injury, air embolism, catheter embolism, thoracic duct injury, nerve lesions, septicæmia, bacteraemia, thrombosis, arterial puncture ...

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CAUTION : Federal law (USA) restricts this device to sale by or on the order of a physician.

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GB) Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections. This device should only be used by, or under the instruc- tion and responsibility of, an appropriately qualified person who is familiar with the product and the clinical procedure.

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