LeaderCuff® (PUR) - Code 1227

Instructions for use

Read all instructions and warnings before use

Product Description

The Vygon single lumen LeaderCuff® has been designed for the percutaneous introduction of a cuffed central venous catheter via the jugular and subclavian veins using a splitable sheath introducer. The compression hub can be removed and replaced by a FloControl® ON / OFF device to facilitate the use of the retrograde tunnel technique (reference 1).

Reference

1 - A cuffed Polyurethane Catheter for Long-Term Central Venous Access : A Novel Technique Prevents Early Displacement. J. Moran & G. L. Sutton; Journal of Parenteral and Enteral Nutrition; Vol. 14, No. 5, 1990

Suggested Insertion Procedure

- Check pack for contents and packaging integrity.
- Use aseptic techniques during the introduction and use of these products.
- Prepare and drape the patient, infiltrate the insertion site and tunnel with local anaesthetic.
- If appropriate place the patient in a moderate Trendelenburg position.
- Using the syringe and introducer needle, locate the vessel and confirm correct location by free aspiration of venous blood.
- Straighten the "J" tip guidewire, gently advance it through the needle into the vessel. Note: The patient's ECG should be monitored to note any rhythm changes indicating over advancement of the guidewire.
- Hold the guidewire in place and remove the introducing needle.
- Make a 0.5 cm incision at the insertion and exit site and undermine with local anaesthetic along the tract of the subcutaneous tunnel.

Anterograde tunnel technique:

- Advance the rounded end of the tunnelling rod along the subcutaneous tunnel to the infraclavicular incision. Attach the catheter to the proximal end of the tunnelling rod.
- Pull the catheter through the tunnel until the cuff is located in the tunnel near the exit site.
- Lay the catheter on the chest wall along the course of the venous system from the insertion site to the superior vena cava.

Warning : During tunnelling allow an adequate loop of catheter at vein entry site to permit normal body movement and reduce kinking.

- Flush lumen of the catheter with heparinised saline.
- Pass the vessel dilator and sheath over the guidewire until the sheath is located in the vessel lumen.

Note : Dilator and sheath must be advanced together or the sheath will split.

- Advance the sheath into the vessel and remove the dilator and guidewire simultaneously, placing a finger over the sheath opening to prevent excessive bleeding or possible air aspiration.
- Pass the catheter through the sheath.
- Carefully pull the sheath apart, peel down to skin level and whilst maintaining forward pressure on the catheter, pull back the sheath and repeat the peeling action. Continue this procedure until all the intravascular catheter length is used and the sheath is totally removed.
- Remove the compression hub and replace with the FloControl® device.
- Check patency by aspirating blood and immediately flush the catheter with heparinised saline. Close the FloControl® device by pushing the switch forward into the OFF position.
- Check the position of the catheter tip by X-ray and close incisions.
- Use the suture wing to provide temporary catheter fixation.
- When the catheter is not in use heparin lock according to hospital protocol.

Product related warnings and cautions

Retrograde tunnel technique :

Catheter

Do not suture directly around the catheter as this will result in damage. Avoid kinking the catheter as this will damage the catheter tubing.

Guide Wire

Never pull back or withdraw the guidewire through a needle. This can damage the wire on the needle bevel. The guide wire should advance without resistance. If resistance is felt do not persist stop and withdraw both the needle and guide wire simultaneously.

Cutting to length

If the catheter is cut to length it should be square cut and not bevelled. Inspect tip profile for suitability before use.

FloControl® ON / OFF device. Instructions for use

The flow through the catheter may be controlled by attaching the FloControl® ON / OFF device. When the switch is pushed forward into the "OFF" position, the fluid path way is closed and when it is moved back into the "ON" position, the fluid pathway is open.

Procedure for attaching or changing the FloControl® ON / OFF device

Place sterile field under end of the catheter.

- Thoroughly clean the catheter and hub.

- Protect the catheter tubing by wrapping gauze around the catheter before applying artery forceps or some form of clamping device.
- Cut catheter next to hub and discard.
- Remove the nut from the compression hub and thread over the end of the catheter.
- Push the catheter onto the nose cone and up to the base of the thread forcing the catheter guide to come out of the housing.
- Withdraw the catheter guide completely and screw the nut fully onto the Vyflow® device.
- Test the security of the catheter on the FloControl® device by holding the catheter tubing firmly and pulling the hub sharply.
- Attach the FloControl® ON / OFF device to the catheter hub, switch OFF and attach a Luer lock plug.
- Unclamp the catheter lumen and flush the line according to hospital protocol.

FloControl® related warnings and cautions

- When using an injection membrane attached to the Vygon FloControl® device, only use short blue or orange needles. Ensure that the FloControl® device is in the "ON" position prior to injecting or aspirating and do not insert the needle more than 8 mm.
- When the catheter is not in use for infusion or aspiration the FloControl® device must be switched "OFF" and a suitable Luer cap locked into the hub. As standard practice, the security of the Luer connections should be checked routinely. This is essential when lubricious substances such as intralipid are being used.

Important Cautions

The using physician should be familiar with and undertake all standard precautions to avoid problems or complications associated with the introduction and use of Seldinger central venous catheters. Regularly check the catheter and catheter hub to ensure that it is not subject to kinking or any undue force or stress. Check that the hub and all IV connections to it are secure and unobstructed.

Sterility guaranteed unless unit pack has been damaged or opened.


Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.


Laboratoires pharmaceutiques VYRON
95440 Ecouen - France

Procedure for attaching the FloControl® ON/OFF device


- 1 - Remove the compression hub from the FloControl® device.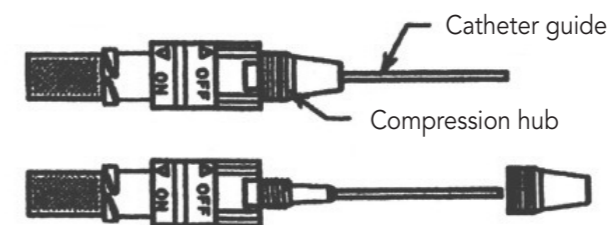


- 2 - Thread the compression hub over the end of the catheter.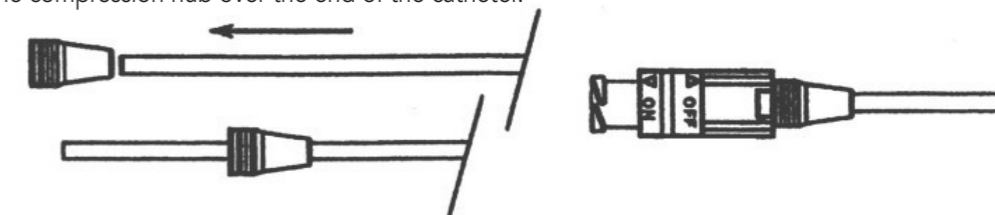

- 3 - Push the catheter onto the nose cone and up to the base of the thread.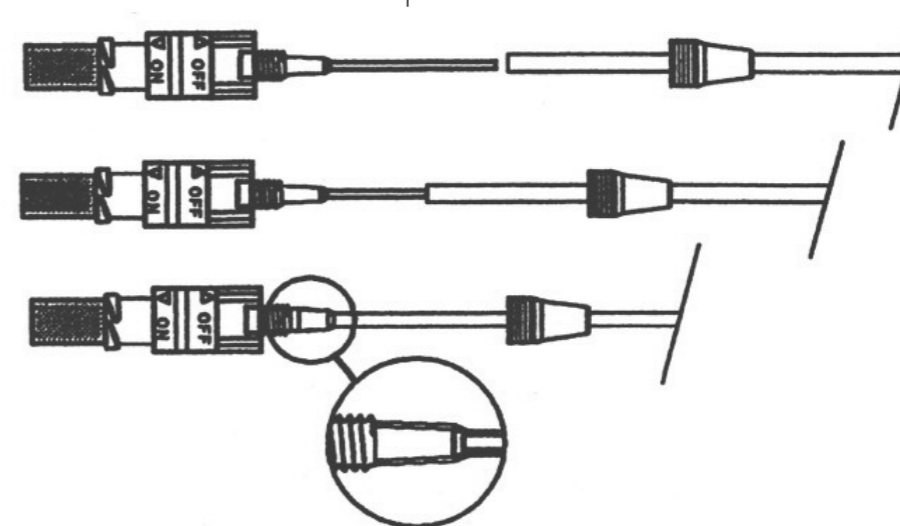

- 4 - Withdrawing the catheter guide, screw the compression hub fully onto the FloControl® device.

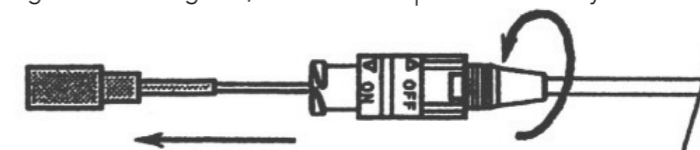- 5 - Completely withdraw catheter guide and test attachment of catheter onto the FloControl® device.

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