B. PERCUTANEOUS TECHNIQUE

This technique involves using the Desilet introducer with peel-apart sheath (code 1146) available separately. It also comes in a kit consisting of a LYFECATH and its introducer.

Important caution : Before placing the peel-apart Desilet, check that the catheter can be introduced without difficulty through the sheath of the Desilet.

- The most frequently used veins are the internal and external jugular or the subclavian vein.
- Prepare (shave) and disinfect the areas close to the puncture point, the tunnelling and the catheter exit site.
- Local anaesthesia at the puncture point of the catheter vein.

• Introduction of the peel-apart Desilet

- Fit the Desilet needle onto a syringe filled with 1 to 2 ml of saline.
- Puncture the vein; passage through the skin layers by the puncture needle fitted to the syringe must be done while maintaining a vacuum in the syringe.
- As soon as blood flows back into the syringe, introduce the "J"-shaped guide into the needle using the "J" straightener. The guide must slide through the needle without difficulty.

Important recommendations : To avoid any risk of the guide accidentally entering the cardiac cavities, do not push this too deeply into the vein. Be careful to leave a sufficient length outside the patient so as to be able to manipulate the guide without risk. **Never pull back or withdraw the guide through the needle. This can damage the wire on the needle level.**

Remove the introducing needle. Maintain hold on guidewire. Make the puncture site larger with a scalpel to permit passage of the dilator and sheath without damaging the tips. Avoid damaging the guidewire at this time. Introduce the assembly vein dilator and peel-apart Desilet sheath onto the guide.

- Slide the assembly along the guide in the vein lumen. This manipulation can be made easier by slight twisting motion of the dilator and sheath on the guide.
- Withdraw the guide leaving the dilator and Desilet sheath in the vein.
- Close the hub of the dilator with the male Luer stopper of the Desilet.

• Tunnelling the catheter

- Fit the distal end of the catheter to the tunnelling needle. In the case of a jugular approach (internal or external) the passage of the tunnelling needle over the clavicle is simplified if care is taken to bend the needle beforehand.
- Tunnel with the needle provided with LYFECATH - the subcutaneous track runs from the catheter exit point to the point of entry of the Desilet into the vein.

- Gently and steadily pull on the tunnelling needle; the catheter follows the tunnelling needle on its subcutaneous track. **The cuff must be placed in the subcutaneous tunnel in the immediate proximity of the exit point** (where the catheter emerges).
- Remove the tunnelling needle.

• Introduction of the catheter

- Estimate the length of catheter needed so that the distal end will be situated at the junction of the superior vena cava and right atrium.
- Cut off the excess length of catheter and flush the LYFECATH with heparinised saline.
- Withdraw the dilator and introduce the catheter into the vein through the Desilet sheath.

- Hold the finger grips on the proximal portion of the sheath, in both hands. Carefully peel-apart the sheath down to skin level. Further advance the catheter, repeat this process until catheter is totally placed and sheath is removed. This operation must be done carefully to avoid withdrawing the catheter from the vein.
- Suture the skin at the entry and exit points of the tunnelling path.

- Carry out a radiological examination to confirm the correct location of the distal end of the catheter, check on the absence of loops and false routing of the catheter.

FIXATION - MAINTENANCE
Warning : Silicone is a delicate material. Pay particular attention when manipulating a sharp or pointed instrument (scissors, scalpel, needle, etc.) in the immediate vicinity of the catheter.

Dressings
 These should be replaced each time they become soiled or lose their adhesiveness or at least once a week in case of intermittent use.

Recommended technique :
 - CAREFUL HAND WASHING.
 - Use sterile gloves and equipment.

- Disinfect the skin with an antiseptic solution at the point of exit of the catheter from the skin, then using a circular motion from the centre towards the outside continue disinfecting, never going over the same place twice with the same compress.
- Carefully dry the skin.
- Fix the section of catheter outside the skin. It is long enough make a safety loop.
- Apply a sterile dressing covering the exit point of the catheter.
- Apply a second occlusive dressing covering the external part of the catheter.

Manipulations of the catheter hubs
 Comply strictly with aseptic conditions each time that the catheter hub is manipulated, particularly when this is advanced the needle too deeply into the injectable latex membrane.

INDICATIONS
 Long-term catheter indicated for :
 - prolonged, permanent or intermittent parenteral feeding, in hospital or at home,
 - chemotherapy during treatment of solid tumours or hemopathy,
 - transfusions of blood products,
 - blood sampling,
 - chemotherapy intensifications and bone marrow transplants
 - various treatments administered to AIDS patients (antiviral drugs, nutrition, etc.).

INSERTION OF LYFECATH
 This catheter is inserted under local anaesthesia in adults and under general anaesthesia in children.
 - Prepare the patient psychologically by explaining the aims and advantages of the technique.
 - Oral premedication is desirable.

- This type of catheter is best inserted into the superior Vena Cava region.
- If possible carry out a local preparation of the implantation site (washing, shaving, cutaneous disinfection of the area which is then covered by sterile drapes) a few hours before introducing the LYFECATH.

There are two insertion methods of LYFECATH :
 A. A surgical technique involving vein exposure.
 B. A percutaneous technique by vein puncture.

Place the patient in Trendelenburg position. Expose the upper thorax, the head turned to the side away from the catheter insertion site.
 Disinfect a wide area around the site chosen for implanting the catheter (area of vein exposure in the case of surgical insertion, site of venous puncture in the case of percutaneous introduction), the tunnelling path and the intended cutaneous exit point of the catheter.

A. SURGICAL TECHNIQUE
 Carry out a local anaesthesia of the skin (at the area of vein exposure and along the tunnelling path).

- Vein preparation
 - Cutaneous incision and dissection of the chosen vein over a length of 2 to 3 cm (cephalic vein in the delto-pectoral groove, external or internal jugular vein in the cervical region).
- Flap two millimetres under the exposed vein and leave.

• Subcutaneous tunnelling and introduction of catheter
 - Make a small incision into the skin at the point chosen for the catheter exit point.
 - Tunnel with the tunnelling needle provided with LYFECATH - the subcutaneous path starts at the catheter exit point and continues as far as the point of introduction of the catheter into the vein (exposed area).
 - Gently and steadily pull on the tunnelling needle; the catheter follows the tunnelling needle on its subcutaneous track. **The cuff must be placed in the subcutaneous tunnel in the immediate proximity of the exit point** (where the catheter emerges).

- Detach the catheter from the needle by cutting the catheter tip with the tunnelling needle.
 - Estimate the length of catheter needed so that the distal end will be situated at the junction of the superior vena cava and right atrium and cut the catheter to the required length.
 - Make a venotomy and introduce the catheter previously flushed with heparinised saline.
 - Check for the absence of subcutaneous kinks or loops on the catheter.

- Remove the siphonets.
- Carry out a radiological examination to confirm the correct positioning of the catheter (location of the distal end, absence of false route, of intravascular loops on the catheter, etc.).

• do not fix the catheter to the vein - such suturing is unnecessary if the catheter is correctly placed (distal end of the catheter at the junction of the superior vena cava and right atrium). The attachment of the catheter to the vein wall could be dangerous when the catheter is removed.
 - Make a double-layered suture of the incision of venous denudation site.
 - Put a spot of heparin on both sides of the catheter exit around the subcutaneous cuff.

REF	Ch/F	Ext.-ø mm	Lumen		L total cm	L intravas. cm	Max. Pressure bar	Desilet introducer
			1	2				
2293.070 / 8293.071	7	2,35	0,6	1,0	93	57	3,5	1146.08
2293.090 / 8293.091	9	3,00	0,7	1,3	93	57	3,5	1146.10
8293.095 / 8293.096	9,5	3,00	1,3	1,3	93	57	3,5	1146.10
2293.095	9,5	3,00	1,3	1,3	93	57	3,5	1146.10
2293.110 / 8293.111	11	3,75	1,0	1,6	93	57	3,5	1146.12
2293.140 / 8293.141	14	4,45	1,6	1,6	93	57	3,5	1146.14

Technique for clearing obstruction from a silicone catheter (blood clot obstruction)
 Clearance with Urokinase :
 - 1500 UI/ml solution.
 - Aspirate, then inject 1 to 2 ml of the above solution from a 10 ml syringe (do not use a syringe with a volume less than 10 ml).
 - After clamping, leave in contact for 1 to 2 hours or even longer.
 - Aspirate the Urokinase.
 - Flush with saline, inject heparinised saline solution.
 (Experience has shown that this technique effectively clears catheters obstructed by a blood clot. It is obviously ineffective against crystals resulting from interactions between incompatible substances).

Warning
 Alcoholics such as ethanol and isopropanol do not change the physical properties of the silicone elastomer if they are used briefly without prolonged contact and if the silicone catheter is then dried.
 The use of iodine tincture and organic solvents such as acetone, chlorinated solvents, ethyl acetate, etc. is inadvisable. These products can degrade or irreversibly deform the silicone elastomer.

Sterile. Sterilized by ethylene oxide. Single use. Sterility guaranteed unless unit pack has been damaged or opened. Discard after single use.
 Caution - Federal law (USA) restricts this device to sale by or on the order of a physician.

Caution: Do not expose this catheter to contact with iodine containing compounds!
 - Remove the siphonets.
 - Carry out a radiological examination to confirm the correct positioning of the catheter (location of the distal end, absence of false route, of intravascular loops on the catheter, etc.).

• do not fix the catheter to the vein - such suturing is unnecessary if the catheter is correctly placed (distal end of the catheter at the junction of the superior vena cava and right atrium). The attachment of the catheter to the vein wall could be dangerous when the catheter is removed.
 - Make a double-layered suture of the incision of venous denudation site.
 - Put a spot of heparin on both sides of the catheter exit around the subcutaneous cuff.

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