MultiCath / MultiStar

Ref 155., 157., 1202., 1203., 1231., 1233., 6202., 6203., 6231., 6233.

INSTRUCTIONS FOR USE

Before inserting the catheter:

1. Prep and drape area surrounding intended venipuncture site, create skin wheal at incision site with local anesthetic agent. To prevent air embolism and to facilitate venous puncture, place patient in an appropriate Trendelenburg position.
2. Assess the length of MultiCath/MultiStar that needs to be introduced; centimeter markings on the catheter assist in correct placement.
3. Identify the different lumens and corresponding hubs of the MultiCath/MultiStar. Match intended use of lumens to patient's clinical requirements, e.g. smallest lumen for low delivery rates. Flow rates for individual lumens of different models are shown on the product pack.
4. All lumens other than the guidewire lumen (distal hub) should be flushed, primed with normal or heparinized saline and kept closed with the injection caps or stopcocks provided.

Catheter introduction:

5. Attach needle to syringe, using a negatively charged syringe, perform venipuncture. Successful venipuncture is confirmed by freeflow of blood into the syringe. (Fig. 1)
6. Remove syringe from needle and attach advancer, advance the guidewire stage by stage until 5 – 10 cm has been inserted.
7. Gently remove advancer from needle hub, hold wire in position and totally remove advancer from wire.

Caution:

Never pull back or withdraw the guidewire while the needle is in place. This could damage the wire on the needle bevel. If any resistance is felt to the advancement of the guidewire, do not persist, withdraw both the needle and the guidewire simultaneously. To further decrease the risk of damage to the guidewire, a short i.v. cannula may be used as an introducer.

8. Holding the guidewire in place, remove the needle. (Fig. 2)
9. If appropriate, the venipuncture site may be enlarged with a scalpel. Take care not to cut the guidewire. (Fig. 3)

10. Dilate the entry point into the vessel by introducing the vessel dilator over the guidewire. (Fig. 4)

CAUTION:

When introducing a catheter or dilator over the guidewire make sure that the length of the exposed guidewire is sufficient so that you can maintain a firm grip on the guidewire at all times. The vessel dilator should not be left in the vessel as an indwelling catheter, to do so should cause vessel damage.

11. After removal of dilator, introduce the distal tip of the MultiCath/MultiStar close and parallel to the skin and progressively advance catheter to the required depth.

12. Holding MultiCath/MultiStar in position at required depth, carefully remove guidewire. Attach a syringe to main lumen hub and aspirate to check that a free flow of blood confirms correct positioning. (Fig. 6)

13. Connect main lumen and any other lumens required at this point.
14. Secure the catheter to the patient using the flexible fixation wing. Spread the wing apart and mount on the catheter from below. The wing position can be changed by again flexing the wing. (Fig. 7, 8)

15. Attach fastener to the wing over the wing groove (Fig. 9) and press firmly together.
16. The wing can then be sutured to the skin. Do not suture directly around the catheter as this may compromise the lumens.
17. Check MultiCath/MultiStar and dressing regularly for evidence of catheter movement, interrupted flow rates, leakage, security of all connections and signs of infection. Particular

care should be taken when changing lines and dressings that the procedure does not damage MultiCath/MultiStar.

Do not kink or use forceps to clamp the lumen tubings during line changes, avoid potential problem of cutting catheter by not using scissors to remove dressings, etc...The clamps on the lead should only be used temporarily. 18. Nursing information available upon request.

CAUTIONS:

Read before inserting and using MultiCath/MultiStar!

1. Correct positioning of the catheter should always be confirmed by x-ray. Correct position will depend on clinical application and introduction point of catheter.
2. The above problems highlight the importance of selecting the most appropriate length of catheter for a particular patient. Left sided neck approaches normally require a longer catheter, e.g. 15 cm length. If longer catheters are used for a right sided neck approach, particular care must be taken not to advance them too far and to ensure there is no accidental migration of the catheter after placement.

Always confirm correct placement by x-ray and observe guidelines on assessing required catheter length and the use of catheter distance markings for both placement and checking for migration. The catheter is graduated 50 mm from its distal tip at 10 mm increments. (except Ref 1202.063/1233.063).

3. Reported complications with catheters are extensive and include hydromediastinum, hydrothorax, catheter embolism, air embolism, thrombosis, laceration of the thoracic duct, temporary paralysis of the phrenic nerve, diaphragmatic paralysis, knotting of the catheter, arterial puncture, bacteremia and septicemia. Follow hospital or departmental protocol to prevent air embolism in catheter placement and subsequent maintenance. Observe strict aseptic technique during placement, maintenance and care of catheter.
4. Carefully clean the insertion site and position drape. The puncture may only be made after the disinfectant has been completely dried on the skin. Be aware that organic solvents such as alcohol or acetone may interact with catheter material and weaken it. Trim catheter to length required, and flush with saline.

5. Only use luer lock connecting tubing with this product.
6. Infusion pressure applied through this catheter should not exceed 1.5 bar.
7. This product may only be sold to or by order of a physician.

8. Do not use this product for monitoring, diagnosis, control or treatment of a defect of the heart or the central venous system.
9. Caution: Do not use small syringes as these can generate very high pressures. It is possible to generate 4 or 5 times the maximum safety pressure, with any size of hand held syringe. Subjecting the catheter to pressure above 21,75psi can result in catheter rupture and embolism. Smaller syringes generate higher pressures than larger ones. Do not exceed a bolus injection pressure of 1,5 bar.

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