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Insertion technique for catheter with peelable cannula

Instructions for use

USA

GB

Should catheter advancement become difficult, infuse a little fluid whilst simultaneously advancing the catheter. This has the effect of dilating the vessel distal to the catheter tip.

Select, prepare, clean and drape site of venipuncture. Apply a tourniquet above the intended insertion site in a manner easily accessible for removal. Remove gloves and don second pair of sterile gloves.

- Remove protective cover from the introducer needle. Using a slow but firm motion, insert introducer needle and sheath into the vein, bevel up, at a 15 to 30 degree angle. Confirm vein entry with blood flashback. Lower the angle of the introducer needle and advance it up the vein.
 - Remove tourniquet. Withdraw the needle while holding introducer sheath stationary. Cover sheath opening with thumb after needle removal to prevent blood backflow and air embolism.
- Note:**
Cannula can be syringe mounted if preferred.
- Insert catheter through cannula using non-toothed forceps using short, steady strokes. Finally advance the catheter to the desired position.
 - Secure the catheter by applying light digital finger pressure on the catheter, beyond the cannula, and slowly withdraw the cannula. Carefully peel the cannula apart whilst maintaining forward pressure on the catheter taking care not to dislodge the catheter from the vein.

Important caution:

At no time should the catheter be withdrawn back through a splitting needle if it becomes impossible to advance the catheter into a satisfactory position, then the needle and catheter must be withdrawn simultaneously. The result of withdrawing a catheter back through the needle can be catheter embolism.

Do not apply direct pressure over the catheter if there is a guidewire in situ during insertion. This may cause the guidewire to become kinked and will make it extremely difficult to withdraw the guidewire from the catheter.

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