



neohelp™

Your unit is using neohelp™ for thermoregulation


Instructions for use

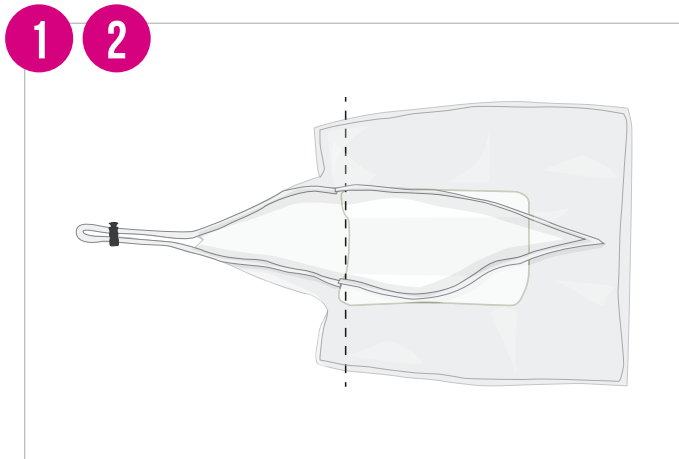


Remove neohelp™ from the packaging and unfold.
Open the VELCRO® hermetic seal ready to receive the baby.

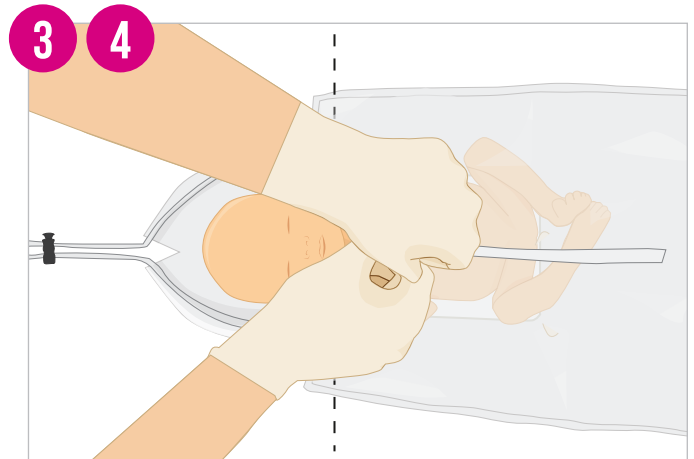

Place baby in neohelp™ (without drying) ensuring shoulders are aligned with edge of foam.
Close with VELCRO® fastener.

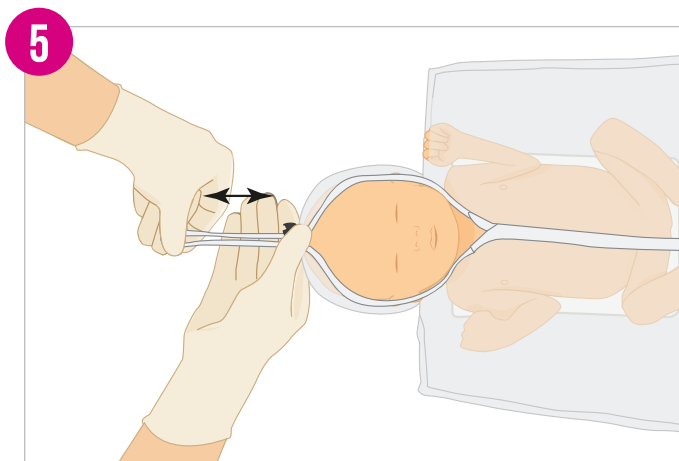

Adjust hood to head.

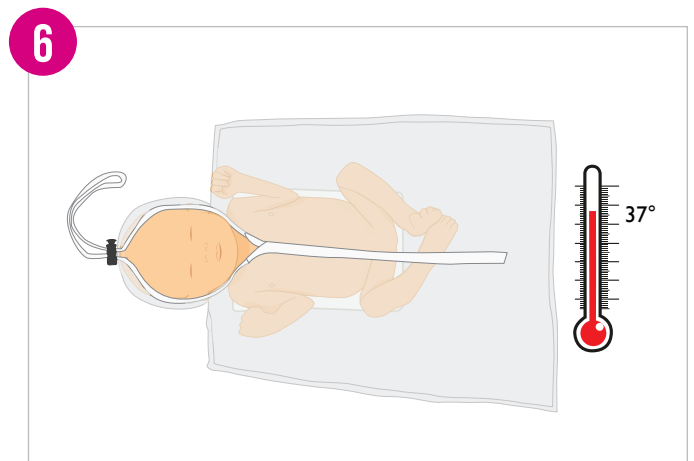

The baby can remain in neohelp™ throughout NICU admission, which will help to maintain temperature and humidity.

Scan for more information

on the neohelp™ neonatal heat loss prevention suit

Please note: The link will take you to an external website

