



Disconnection and flush of a pump secured with Qimono at the end of treatment

Disconnection and flush of a pump secured with Qimono at the end of treatment

1 Disconnect

Disconnection and flush of a pump secured with Qimono at the end of treatment



Qimono closed system is a universal luer lock connection - enabling any luer connection both on the patient and infusion side

2 Flush

Start flush according to local protocols



3 Access & Dressing

Continue with standard clinical practice with your choice of vascular access device

4 Disposal

Dispose of the full system in the clinical waste bin.



Qimomale and Qimofemale are sterile medical devices class IIa, compliant with 93/42CEE directive as to appendix 2. Certification established by G-MED, notified body N°0459. Medical devices manufactured and distributed by Vygon.