


REMOVAL STEPS

SAFETY HUBER NEEDLE

PPS **Flow +**® Automatic positive pressure

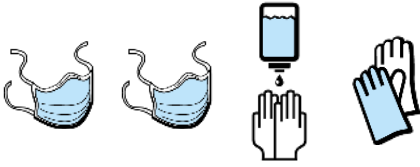


- Remove the dressing by stretching it parallel to the skin
- Remove the adhesive strips
- Remove the foam cube
- Flush the needle with saline using a push/pause flushing technique.




1

Hold the blue platform with 2 fingers.

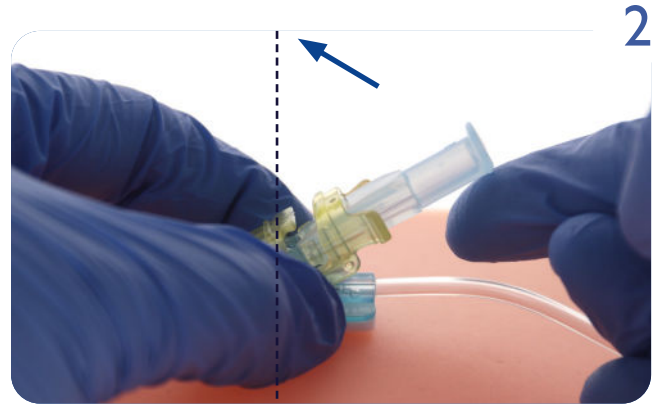


2

Lift the piston from the horizontal position to the vertical position (perpendicular to the septum) while holding the platform with 2 fingers.

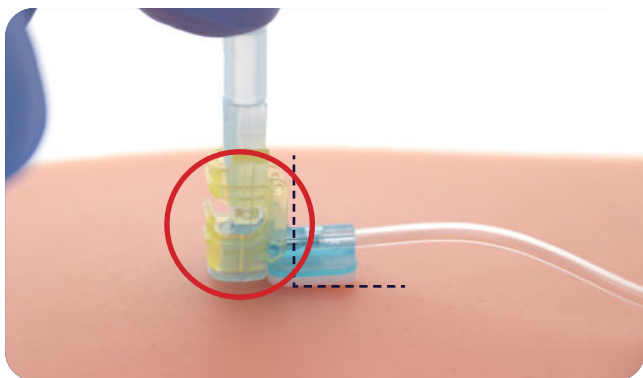


3

Make sure, the needle piston is perpendicular to the septum and in contact with the skin.

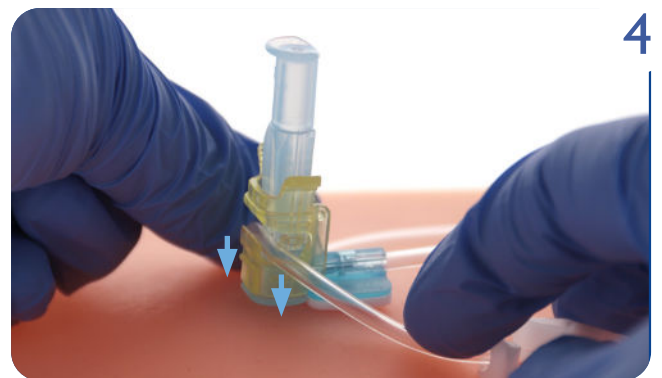

4

Insert the line into the notch and pull it downwards.

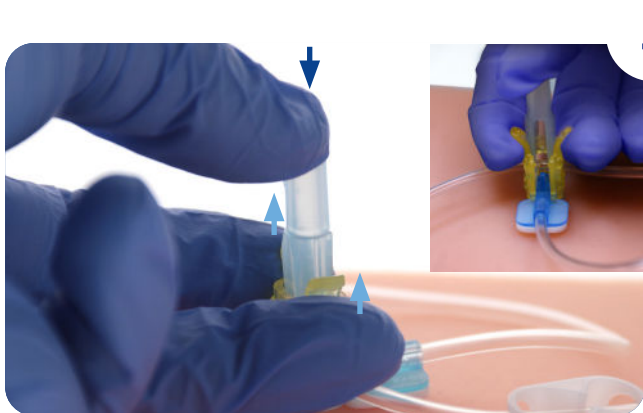

5

Place your hand on the opposite side of the blue platform, put your thumb on the piston, your forefinger and middle finger under the winglets (on both side of the piston).

Maintain the piston in the horizontal position with your thumb and lift the extractor (with an upward movement) until there is a **CLICK** corresponding to the total locking of the needle.

This final step eliminates the risk of accidental needlestick injury.



Pressure of piston on tubing during removal leads to automatic injection of saline solution into the implantable port = **POSITIVE PRESSURE**



Value Life



Video available here